Nu Radiance[®]

Classic[™]

Teeth Whitening System
Dentist - Supervised Time-Released Gel

Patient Instructions

Manufactured by Nu Radiance, Inc.
Green Bay, WI 54304

Active Ingredient: Carbamide Peroxide

Nu Radiance® Classic™

Patient Instructions

Your dentist has just provided you with one of the most advanced teeth whitening systems available.

Wear the tray as directed by your dental professional!

Nu Radiance provides a pleasant, dental professional supervised, home-bleaching system. The gel consists of the active ingredient carbamide peroxide, suspended in a high tack, high diffusivity, amine free gel of near neutral pH, providing optimum whitening results with reduced sensitivity.

1. Rinse mouth prior to insertion of bleaching tray. Pat your teeth dry with tissue prior to inserting tray.
2. Load the tray with bleaching gel. Remove cap from the pre-loaded syringe, apply small drops on the portion of the tray that will contact the teeth. Do not overload the tray.
3. Place the tray firmly into position over the teeth. With a tissue, wipe off any excess gel. Wear the tray 1-4 hours per day or as prescribed by your dentist. You can wear the tray for as little as 30 minutes at a time, twice a day and have excellent results. While wearing the tray do not eat, smoke or drink.

To obtain the most rapid results, it is important to follow the usage routine that your dentist recommends. The whitener is effective for up to four hours, however most whitening occurs in the first hour. The longer your teeth are exposed to the bleaching gel, the faster they will lighten.

If you experience any problems please contact your dentist's office.

Warning: This product should not be used by children under 12, or pregnant or nursing women. Do not use if you are allergic to any of the ingredients listed. Do not expose whitening gel to heat or direct sunlight. Maintain freshness by storing in a cool, dry place.

Ingredients: Glycerin and/ or propylene glycol, carbamide peroxide, buffered polyacrylic acid and peppermint oil.

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