Nu Radiance[®]

Duet[™]

**Dentist Dispensed
30 Minute Treatments
Sensitivity Free**

Teeth Whitening System

Patient Instructions

Nu Radiance® Duet™

Your dentist has just provided you with one of the most advanced teeth whitening system available.

Nu Radiance® Duet Sensitivity Free formula hydrates the teeth and thus does not cause sensitivity. It is one of the most advanced formulations for safe, effective and non-painful teeth whitening. The Duet gel is a carbamide peroxide gel mixed with an accelerating and hydrating agent to speed up the whitening process while essentially eliminating tooth sensitivity.

Read instructions before use.

Always use a mixing tip on the syringe. After each use place the tip-guard over the tip on the syringe until the next use.

Pat teeth dry before whitening. For long term storage, store in a refrigerator.

PATIENT INSTRUCTIONS

1. Place the mixing tip onto the syringe and twist to lock into place. Do not pull back on the plunger, this can cause activation of the gel within the syringe.
2. Place small drops of freshly mixed gel on that portion of the tray that will contact your teeth. Use the gel sparingly. As the gels are released they will be blended by the mixing tip and activated. After each use place the tip-guard over the tip. The tip-guard will contain a small amount of gel that remains active within the tip. When ready for the next use remove the tip-guard and expel a small amount of gel about the size of a pea. The gel that was in the mixing tip from the last use is no longer active. Do not pull back on the plunger as it could cause the activation of the gel.
3. Place tray into mouth seating the tray against the teeth to be whitened. Remove any excess gel with tissue. If there is any excess gel, reduce the quantity in subsequent sessions.
4. After about 30 to 60 minutes remove tray and rinse with water.
5. Whiten your teeth twice a day or as directed by your dentist. Do not eat, smoke or drink while whitening or for 30 minutes after.

Each of the dual syringes has about 5-6 applications for both your top and bottom teeth.

WARNING: This product should not be used by children under 12 or pregnant or nursing women. Do not use if you are allergic to any of the ingredients listed.

Ingredients: Propylene glycol and/or glycerin, water, carbamide peroxide, silica, buffered polyacrylic acid, xylitol, sodium carbonate, cellulose gum, Germaben II, peppermint oil, FDA accepted dyes.

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