# Nu Radiance® Forté®

Whitens & Renews With Calcium

Forté Teeth Whitener, which contains calcium peroxide, rebuilds enamel with significant calcium uptake into incipient lesions while also increasing the surface hardness, for healthier teeth.

This 30-minute take home system whitens teeth and gives them a smooth clean feeling with a whiter, brighter, more lustrous looking smile.

## PATIENT INSTRUCTIONS

Pat teeth dry before whitening. For long term storage, store in a refrigerator.

Each dual syringe has about 5 - 6 applications for both your upper and lower teeth. Each syringes comes with 6 mixing tips. Do not pull back on plunger as it could cause activation of the gel.

### Dual Arch Application

After each use remove mixing tip and replace syringe cap. When ready for the next use, remove syringe cap and put a new mixing tip onto syringe.

### Single Arch Application

After each use remove mixing tip and save tip. Replace syringe cap. When ready for the next use, reuse saved mixing tip. If tip becomes clogged, discard mixing tip and use fresh mixing tip.

1. Place the mixing tip onto the syringe and twist to lock into place. Do not pull back on the plunger; this can cause activation of the gel within the syringe.
2. Use the gel sparingly. Place a small amount of freshly mixed gel in the front part of the tray (that area of the tray which lies against the front of your teeth). As the gels are dispensed they will be blended by the mixing tip and activated.
3. Place tray into mouth, seating the tray against the teeth to be whitened. Remove any excess gel with a tissue.
4. After about 30-minutes to 1 hour remove tray and rinse with water.
5. Whiten your teeth once or twice a day or as directed by your dentist. Do not eat, smoke or drink while whitening.

**WARNING:** This product should not be used by children under 12 or pregnant or nursing women. Do not use if you are allergic to any of the ingredients listed.

Active Ingredients: carbamide peroxide and calcium peroxide also contains: water, propylene glycol and/or glycerin, sodium pyrophosphate, poly-ether, calcium carbonate, silica, titanium dioxide, phosphoric acid and peppermint oil.

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**Nu Radiance, Inc.**