EVA
Tray Forming Instructions
For
Nu Radiance[®]
Teeth Whitening Products

Classic[™]

Duet[™]

Forté[®]

Nu Radiance, Inc.
795 Coronis Way, Green Bay, WI 54304
Telephone: 920/965-0953
Fax: 920/965-0954
www.nuradiance.com
E-mail: info@nuradiance.com

EU Authorized representative:
Quality First International Limited
Channelsea, 3rd Floor Canning Road, Strat-
ford, London, E15 3ND, United Kingdom
Telephone: +44 20 8 522 1937
Fax: +44 20 8 522 1937
E-mail: Hard616@aol.com

ISO 9001:2001, ISO 13485:2003

CE 0510

***EVA Tray Forming Instructions
For
Nu Radiance®
Teeth Whitening Products***

Step 1 - Prior to bleaching be certain gingival tissue is healthy and there are no areas of caries. Bleaching is best if there is minimal calculus and superficial stains, so a thorough cleaning prior to initiating the bleaching campaign is recommended. When bleaching prior to resin bonded restorations, allow sufficient time (about 2 weeks) between the final bleaching session and initiating the resin bonding.

Step 2 - After the impression has been taken, pour model with fast set plaster or dental stone. When using alginate, pouring shortly after impression is advisable. Make sure model is free of bubbles to assure a proper fitting bleaching tray. Trim model so that the base is exposed and excess stone around the model periphery (beyond 5 mm) is removed. This will make trimming the tray easier.

Step 3 - Soften the EVA tray material in a vacuum former until the material is uniform and sagging down about 1/2”-1”. Bring EVA material over model and then turn on the vacuum switch. Allow plastic sufficient time to cool before handling.

Step 4 - Trim plastic from around model periphery. A non-serrated curved crown and bridge scissor works best for final model trimming. Trim the tray tight to labial and lingual margins, with no more than 1mm tray extension onto tissue. (Scalloping tray over inter-dental papilla may be necessary for patients who are prone to tissue sensitivity or who develop soreness on gums due to the bleaching agent.)

Step 5 - Fine tune trimming to eliminate jagged edges on tray. Use the model as a guide to assure custom fit.

Step 6 - Deliver the tray to the patient and carefully explain how to put the solution into the tray and how to insert the tray into the mouth. It is a very easy procedure for patients to follow and thus patient compliance is high. It is also advisable to review the patient instructions with the patient.

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