



Nu Radiance

# Insty-Tray

## Instant wax tray

Instructions: For both upper and lower arches. Place a line of gel just to the front of an imaginary center line of the tray. Insert the tray over the teeth and close your mouth, holding the tray against the front of your teeth with your lips. As the tray softens (about 5 seconds) fit to your teeth using your tongue and lips. Insty-Trays are reusable, just rinse with cool water and reapply gel.

If there appears to be too much length at the front of the tray, simply fold a little over and pinch while fitting.

The variability of the tray edge is part of the manufacturing process and does not affect the tray's performance.

Patent Pending  
US FDA Food Grade Wax  
Made in USA

Nu Radiance, Inc  
Green Bay, WI 54304